

Gray Lid Seal , 6-foot roll

P/N 53007 / 429614

Ozark Biomedical – (800)457-7576 – www.ozarkbiomedical.com

The OEM Lid Seal Retrofit Kit, P/N 429614, was discontinued by Becton Dickinson in 08/2020. The primary component of the kit, the gray foam seal, functioned as an upper seal attached to the lid assembly of the SeroFuge 2000 Series centrifuges.

The product, P/N 53007, is 6' of gray foam gasket material that offers a suitable replacement of the OEM seal. This foam seal must be trimmed to fit the necessary application, and requires minor surface preparation to ensure proper adhesion.

Surface Preparation Instruction

The installation surface for the lid seal must be cleaned of any remaining defective gasket material and any residual adhesive material. Once cleaned, the painted surface of the lid, only where the gasket is to be installed, needs to be rubbed gently with a Scotch-Brite® pad or similar abrasive material and then cleaned with isopropyl alcohol. This mechanically prepares the surface for good adhesion of the foam seal's adhesive backing.

Once surface preparation is complete, the new foam seal can be cut to fit the layout of the OEM's original seal. Removing the paper backing of the foam seal reveals the adhesive and can then carefully installed on the lid assembly. Any unused foam seal can be discarded when finished.

